

ENDORACTOR[®]

New Ideal Endoscopic Sponge
for Retraction of small intestine

health **care** partner GmbH

ENDORACTOR[®]

Key Features

No need of removing organs by an extreme head down position

+ Endoractor enables the surgery in an early horizontal position and reduce the risk.

Reduce stress of surgeons from descent of organs

+ Using Endoractor secures a good surgical field during operation once excluded and reduce the stress of surgeons.

Enables wide range exclusion of organs

+ Swells approximately 9 times.

Reduce risk of damage of organs

+ Less tissue trauma made from cellulose material and reduce risk of damage of organs compared to other retractors.


X-ray detectable

+ Reduce risk of internal residue.


Wide center & curvature design makes effective exclusion of the small intestine.

+ (for Type J 20cm)

Use Procedure

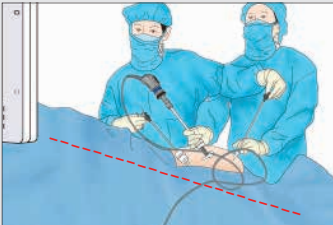


Place a patient in a head-down position or other necessary position to exclude organs other than the target.

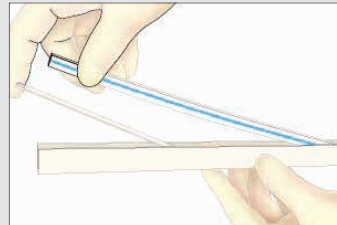


Take out the product from the sheath.



Insert the product from a trocar with diameter not less than 12 mm.

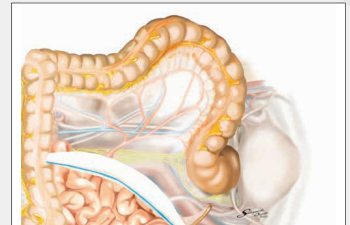

Place the product at an appropriate location and spray physiological saline to the product as swell.

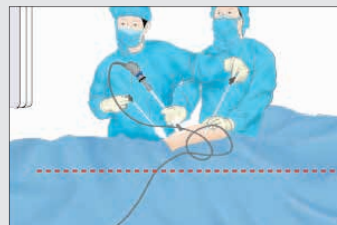

Return the patient's position to a nearly horizontal position and conduct a surgery.



Make a small incision and take out the product from the body cavity.

Product Code	Specification	QTY/Box
088-480000-02	24 cm	1 pc / pack, 6 packs / box, 2 boxes / carton
088-480010-02	Type J 20 cm	1 pc / pack, 6 packs / box, 2 boxes / carton